

## IV. Product Usage

1. Remote Control Buttons
2. Temperature Setup
3. Usage Methods per Mode
  - ① Automatic Mode Use
  - ② Semi-Auto Mode
  - ③ Manual Mode
  - ④ Massage Mode
4. Methods of Product Use

# 1. Remote Control Buttons

## Current Temperature Indicator

This light turns on when the current temperature is displayed

## Set Temperature Indicator

This light turns on when the set temperature is displayed

## Internal Projector Indicator

This feature indicates that the internal projector is selected

## External Projector Indicator

This feature indicates that the external projector is selected.

## ON/OFF Button

This button turns the power on or off

## Message Mode Indicator

This feature indicates that the message mode is selected

## Message Select Button

This button is used to select the message mode

## Temperature Display Window

Displays the current temperature of the selected device

## Temperature Setup Button

- \* This button selects the device to set the temperature
- \* Pressing the button for 3 seconds will display the current temperature

## Up Button

This button increases the temperature or moves the internal projector up

## Down Button

This button decreases the temperature or moves the internal projector down

## START/PAUSE Button


This button starts or pauses operation (auto, semiauto, message)

## AUTO/SEMI Indicator

This feature indicates the AUTO or SEMI mode selected

## AUTO/SEMI Button

This button selects the auto, semi, or manual modes.

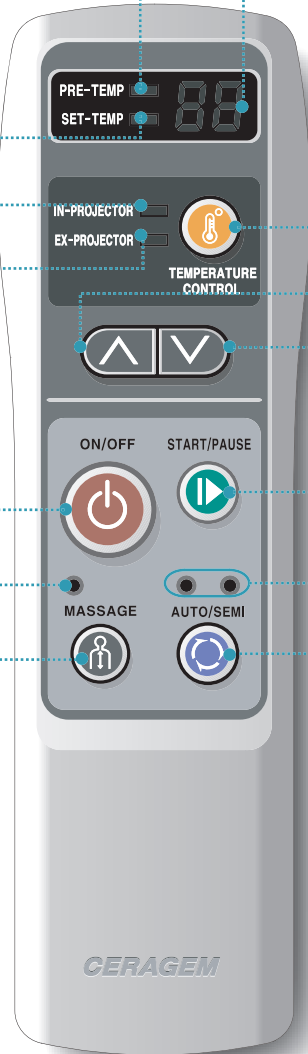

## ■ Error Messages

(If an error occurs, the error message will be displayed in the "Temperature Display Window" of the Remote Control)

Indication	Meaning	Location and Info of Error LEDs	
		LED Location	Info
SE	Sensor error	Sensor error	Internal projector LED
		3-Sphere projector LED	3-Sphere projector sensor malfunction
EH	High temperature	Internal projector LED	High temperature in internal projector
		3-Sphere projector LED	High temperature in 3-Sphere projector
CE	Communication error		Communication error
LE	Limit switch error	Internal projector LED	Error in the main mattress limit
od	Motor error	Internal projector LED	Main motor error



### **WARNING**

When an error message is displayed, turn off the remote control and the power switch on the side of the product for 5 minutes. Continued errors should be reported to a nearby service center.

## 2. Temperature Setup

**1** Select the internal or external projector by pressing the temperature setup button.


- Select between internal or external projectors
- When the button is pressed, the Set Temperature Indicator will blink



**2** Adjust the temperature with the UP/DOWN button which will be configured after 3 seconds.

- The range of the temperature setting is 30~60°C adjustable increments of 5 °C

(The product will resume to the configurations from its previous use)

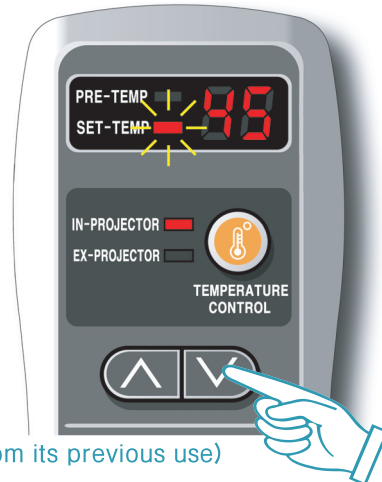

**TIP** How do I turn off the heating feature of the internal and external projectors?

To turn off the heat for the selected device, press the (U) button when the "Temperature Display Window" starts to blink.

※The internal projector can also be used as a massager without thermotherapy