



4. Remote control error messages

In case of error during operation, the following messages appear on the remote control display:

Symbol	Meaning
E1	When the internal projector's temperature is over 75°C, the "E1" error appears on the temperature display of the internal projector, and all operations of the product stop.
E2	When the external projector's temperature is over 70°C, the "E2" error appears on the temperature display of the external projector, and all operations of the product stop.
E	If the motor is overloaded, and certain abnormal current is detected, the "E" error appears on the automatic mode display, and all operations of the product stop.
C	If normal communication between the remote control and the main body is impossible, the "C" error appears on the automatic mode display, and all operations of the product stop.
LE	It appears when the limit switch is faulty and all operations of the product stop.



Caution

After checking the error messages, contact the Customer Support Center.